

Product Name: 2Cool™ Zimmer Chiller Adapter

Part Number: 1500-289-00

Description: The 2Cool Zimmer Chiller Adapter provides the ability for the user to divert a portion of the cool air from the chiller directly to the patient via a second hose that is attached via the “Y” shaped adapter. As pictured in Step 1.

Directions for Use:

Step 1: Attach the locking end of the hose onto one side of the 2Cool adapter.

Step 2: Attach the single end of the adapter onto the chiller as shown below in step 2. The adapter diverts a portion of the air to the HALO™ handpiece and a portion to the patient therefore you may need to adjust your Zimmer settings based on the needs of your patient.

Step 1



Step 2

